



V2T-10

Voice to Text converter for landline phone

Convertisseur Voix en Texte pour téléphone fixe

Convertidor voz a texto para teléfonos de líneas fijas

Sprache-zu-Text Umwandler für schnurgebundene
Telefone



English P1

Français P14

Español P28

Deutsch P41

Congratulations on purchasing your Geemarc V2T-10. The V2T-10 is an assistive listening device working with **Text Hear Personal Hearing Aid** App running on your smartphone or tablet. It converts the speech to text and at same time you make/receive phone calls through your telephone line.

It is important that you read the instructions below in order to use your Geemarc V2T-10 to its full potential. Keep this user guide in a safe place for future reference.

Unpacking the V2T-10

When unpacking the V2T-10, you should find the following in the box:

- V2T-10 Piece.
- 9V ALKALINE BATTERY.
- RJ9 TO RJ9 Cord.
- 3.5mm Plug to 3.5mm Plug Audio Cord.
- 1 User Guide



For product support and help visit our website at

www.geemarc.com

Telephone **01707 387602**

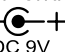
Or fax **01707 832529**

Setting Up – How to set the battery or AC-DC power Adaptor

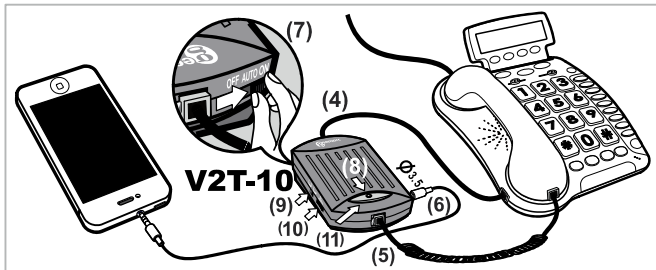
Open the battery compartment by pressing the small plastic lock on battery cover **(1)** located on the rear of the V2T-10 Piece. Insert the supplied battery (9V PP3) into the battery compartment **(2)**, and then replace the battery cover.

NOTE: Insert battery with the correct polarity as shown in the battery compartment.

A 9V AC-DC Power Adaptor (not included) can be used to power the V2T-10 unit.



Plug the Power Adaptor into the mains supply and plug the other end of the lead into the DC Power Supply Port **(3)** () on the right hand side of the V2T-10. Switch the mains supply **on**.

NOTE: Make sure the correct polarity of Adaptor output plug, centre is “+”.



Setting Up – How to Connect the V2T-10 to the Telephone


To connect your phone to the V2T-10:

- Unplug the Handset cord from your telephone base socket.
- Connect one end of RJ9 to RJ9 cord supplied with the V2T-10 to the telephone base socket and one end to the input socket **(4)** () located at the top of your V2T-10 .
- Connect your handset cord to the output socket **(5)** () located at the front of your V2T-10.

IMPORTANT: The V2T-10 cannot be used with one-piece telephones which have a keypad on the handset.


Setting Up – How to Connect the V2T-10 to the Smart-Phone or Tablet

To connect your smart-phone or tablet to the V2T-10:

- Connect one end of 3.5mm to 3.5mm cord supplied with the V2T-10 to the 3.5mm socket of your smart-phone or tablet and one end to the Audio output port **(6)** () located at the right hand side of your V2T-10.
- Download our **Text Hear Personal Hearing Aid** App from **Google play** and run it on your smart-phone or tablet.

IMPORTANT: At this stage, The V2T-10 only can be used with smart-phone or tablet running Android system.

Turning the V2T-10 On and Off

Put the mode switch (7)  to **ON** position, The Power On LED (8) will display RED, and the V2T is ready to work.

To turn the V2T-10 off, put the mode switch (7) to **OFF** position. The RED LED will switch off.

AUTO mode: Put the mode switch (7) to **AUTO** position, The Power On LED (8) will display RED, if there has no phone call for a certain minutes, the V2T will auto power off to save battery life. When there is a phone call, the V2T-10 resumes automatically.

Battery Indicator

The Battery indicator (8) should illuminate every time you use V2T-10. If the LED start to fade or does not light when your V2T-10 is in use, you should change the battery to ensure optimum performance.

Making or Answering a phone call

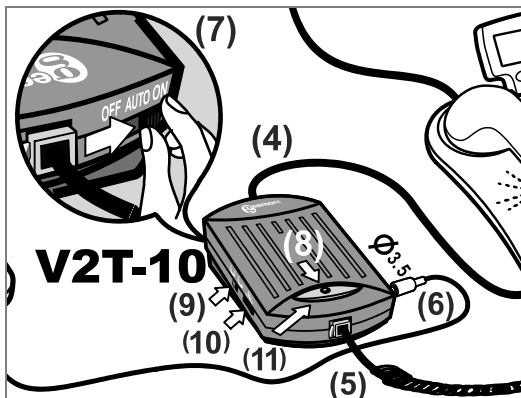
When you want to make or answer a phone call, just do it as you do with your telephone!.

IMPORTANT: If you don't use the V2T-10 with Text Hear Personal Hearing Aid App, You can turn the V2T-10 off to save battery life. Turning V2T-10 to off will not affect your phone calls.



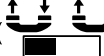
Voice to Text Convertor

- Working with Text Hear Personal Hearing Aid App on your Smart-phone or Tablet

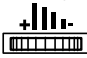
- Download and run **Text Hear Personal Hearing Aid APP** on your smart-phone or tablet.
- Connect the V2T-10 to your smart-phone or tablet as instructed above.
- Put the V2T mode switch (7) (OFF AUTO ON) to ON or AUTO position.
- When you make, or answer a phone call, the speech will be converted to Text in **Text Hear Personal Hearing Aid APP** automatically.



- Select and Convert Voices to your Smartphone or Tablet

You can convert **your voices to texts** () or **both you and friend's voices to text** () using the voice convert source selector (**9**) () at the left side of V2T-10.

- Adjust Voice Volume send to your Smartphone or Tablet

You can adjust the volume send to your Smart-phone or tablet by turning the wheel (**10**) () at the left side of V2T-10. Adjust the volume to get a best performance in the APP.

- MUTE function during voice to text converting

If you don't want to convert the voice to text during phone call, just press and hold down the **MUTE button (11)** on the top of the V2T-10